

mm	60	76	91	101	111	121	126	136	141	151	161	173	186	198
	75	90	100	110	120	125	135	141	150	160	172	185	197	207
Size	00	0	1	2	3	4	4.5	5	6	7	8	9	10	11
in	2.3	3.0	3.6	4.0	4.4	4.8	5.0	5.4	5.6	6.0	6.3	6.8	7.3	7.8
	2.9	3.5	3.9	4.3	4.7	4.9	5.3	5.5	5.9	6.2	6.7	7.2	7.7	8.1



Step 1:

Clean the hoof and measure either the length and width of the hoof as shown in the diagram to the right.

If due to an underlying condition there are significant differences in the length and the width of the horse's hoof, always take the larger dimension. This also applies if the measurements of both front or hind feet differ between left and right, the bigger mould will fit both. Consider, that the mesh will add approx. 2 mm depending on the amount of layers used in the applicatoin process.


Step 2:

Check the FormaHoof sizing chart and select a size range which fits with your measurement. Do not include bulbs of the heel in the measurement and select the correct units in the table.

The Shape Question - Regular or Slimline?

In order to achieve the perfect fit of your FormaHoof application, you must measure your horse's feet and determine whether you need a slimline or regular Mould.

- The sole **length** is the same on each model and shape.
- The **width** on reusable slimline Moulds is narrower than the length provided and will fit horses with narrow feet better. (See samples on the right)

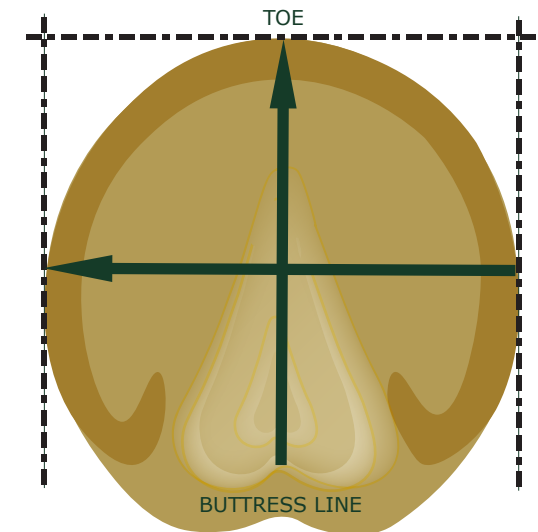

REGULAR



SLIMLINE